Slope Gas

Contenu

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Slope Gas Description

| | Volume | Ingredients | Storage | Stability |
|--|--|--|---------|---|
| | 20 Std. Cu. Ft. (560 Std. Liters) 2200 psig @ 21°C (70°F) | 10% Carbon Dioxide, 0% Oxygen and Balance Nitrogen | 18-25°C | Until expiration date on the gas tank label |

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Disposal of Empty Gas Tanks

- Using a flush type valve wrench, close the gas tank valve stem (A).
- Disconnect the gas regulator from the gas tank by unscrewing the yoke handle (yoke screw).
- Remove the gas tank to a well ventilated, open area.
- Position valve outlet (B) face down, away from people and loose objects.

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- Using a flush type valve wrench, release the gas tank contents by slowly turning the valve stem (A) counterclockwise.
- When the gas tank is completely vented and pressure reduced to zero, label the container 'EMPTY' and dispose according to your laboratory protocol.

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| Symbol | Description |
|--------|---|
| | <i>In vitro</i> diagnostic device |
| REF | Catalog Number |
| | Manufacturer |
| | Authorized Representative |
| CE | CE Mark |
| | Consult operating instructions |
| | Attention, see instructions for use |
| | Temperature limitation (Store between 18°C–25°C) |
| LOT | Batch code |
| | Use by |
| | Pressure |
| | To recycle this cylinder in the US (Cylinder Recycling Service) |
| | Non-flammable gas |

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Slope Gas

Contents

| REF | Contents |
|----------------------|--|
| 03294747 (477438) | 1 Gas Tank of Gas Mixture: 10% CO ₂ , 0% O ₂ , Balance N ₂ Gravimetric standard, accuracy ±0.03% |

03213-2

Intended Use

For *in vitro* diagnostic use as a gas standard for calibration of Bayer HealthCare blood gas analyzers and systems.

Slope Gas Description

| | Volume | Ingredients | Storage | Stability |
|---|--|--|---------|---|
|  | 20 Std. Cu. Ft. (560 Std. Liters) 2200 psig @ 21°C (70°F) | 10% Carbon Dioxide, 0% Oxygen and Balance Nitrogen | 18–25°C | Until expiration date on the gas tank label |

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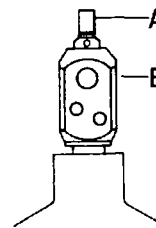
Instructions for Use

CAUTION: Contents under pressure. Do not puncture or use near an open flame.

Refer to the appropriate Bayer Instruction Manual or Operator's Manual for detailed instructions about the gas regulator, and gas pressure and flow adjustment.

- Remove the plastic seal from the valve assembly of each gas tank.
- Verify that the gas tank seal (sealing washer) is in place on the gas regulator.
- Attach the gas regulator to the gas tank by aligning the yoke inlet (regulator nipple) with the valve outlet (B).
- Firmly tighten the gas regulator yoke handle (yoke screw).
- Using a flush type valve wrench, open the gas tank by turning the valve stem (A) counterclockwise.
- At the end of each use, close the gas tank by turning the valve stem clockwise.

Outlet as recommended by "Compressed Gas Association" (U.S.)

Disposal of Empty Gas Tanks

- Using a flush type valve wrench, close the gas tank valve stem (A).
- Disconnect the gas regulator from the gas tank by unscrewing the yoke handle (yoke screw).
- Remove the gas tank to a well ventilated, open area.
- Position valve outlet (B) face down, away from people and loose objects.
VAROITUS: Vältä joutumista kosketuksiin kaasuvirtauksen kanssa. Paineistettu kaasu voi aiheuttaa loukkaantumisen tai omaisuusvahinkoja.
- Using a flush type valve wrench, release the gas tank contents by slowly turning the valve stem (A) counterclockwise.
- When the gas tank is completely vented and pressure reduced to zero, label the container 'EMPTY' and dispose according to your laboratory protocol.

Traceability of Standardization

Gas Calibration Standards are traceable to NIST using gravimetric weighing methods calibrated against Class S weights.

Warranty

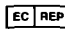
Bayer provides products which, under normal conditions of use, meet stated specifications. Gas products should be utilized only with equipment constructed of compatible materials. Gases under pressure present inherent hazards to the user and to equipment, and it can only be assumed that the user has basic knowledge in the use and limitations of the products. Field repairs to equipment or the refilling of a gas container by the user automatically void any manufacturer's liability.

Technical Assistance

For customer support, please contact your local technical support provider or distributor.

Understanding the Symbols

The following symbols may appear on the labeling and packaging:

| Symbol | Description |
|---|---|
|  | <i>In vitro</i> diagnostic device |
| REF | Catalog Number |
|  | Manufacturer |
|  | Authorized Representative |
| CE | CE Mark |
|  | Consult operating instructions |
|  | Attention, see instructions for use |
|  | Temperature limitation (Store between 18°C–25°C) |
|  | Batch code |
|  | Use by |
|  | Pressure |
|  | To recycle this cylinder in the US (Cylinder Recycling Service) |
|  | Non-flammable gas |

4508CC ED

Manufactured for Bayer

Bayer HealthCare LLC
Subsidiary of Bayer Corporation
East Walpole, MA 02032-1597 USA

Bayer Diagnostics Mfg (Sudbury) Ltd
Sudbury, CO10 2XQ, UK

Bayer S.A.
División Diagnósticos
Ricardo Gutiérrez 3652
B1605EHD Munro - Buenos Aires
Argentina
+54 (0) 11-4762-7000

Bayer Australia Limited
ABN 22 000 138 714
Diagnostics Business Group
2 Keith Campbell Court
Scoresby Victoria 3179
Australia
+61 (0) 3-9212-8444

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Bayer Inc.
Bayer Diagnostics
Healthcare Division
77 Belfield Road
Toronto, Ontario
Canada M9W 1G6
416-248-0771

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Bayer Hellas S.A.
Akakion 54A
Marousi
Athens 151 25, Greece
+30 (0) 1-6883648

Bayer HealthCare Co. Ltd
20/F Gee Chang Hong Centre
65 Wong Chuk Hang Road
Hong Kong
+852-28147337

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Bayer New Zealand Ltd
Diagnostics Business Group
3 Argus Place,
Glenfield,
Auckland, New Zealand
+64-800-724-269

Bayer As
Brennaveien 18
N-1483 Skytta, Norway
+47-67-06-86-00

Bayer Sp. Z o.o.
Al. Jerozolimskie 158
02-326 Warszawa, Polska
+48 (0) 22-572-3500

Bayer Diagnostics Europe Ltd
(sucursal em Portugal)
Rua Quinta do Pinheiro, 5
2795-653 Carnaxide, Portugal
+351-21-416-4227

Bayer Puerto Rico Inc.
Diagnostics Division
Victoria Industrial Park
Building 1
Carolina, Puerto Rico
787-752-8989

Bayer (South East Asia) Pte Ltd.
No. 9 Benoi Sector
Singapore 629844
+65-261-3389
Bayer (Pty) Ltd.
Healthcare Division
27 Wrench Road
Isando 1600, South Africa
+27 (0) 11-921-5048

Química Farmacéutica Bayer, S.A.
División Diagnósticos
Calabria, 268
08029 Barcelona, España
+34-93-4956500

Bayer AB
Drakegatan 1
S-402 24 Göteborg, Sweden
+46-31-83-98-00

Bayer (Schweiz) AG
GB DS
Grubenstrasse 6
CH-8045 Zürich
+41 (0) 1-465-81-11

Bayer Taiwan Co. Ltd.
Diagnostics Division
11/F 237 Sung Chiang Road
Taipei 104
Taiwan
+886-2-2-5039123

Bayer plc
Diagnostics Division
Bayer House
Strawberry Hill
Newbury, RG14 1JA
United Kingdom
+44 (0)1635-563000

Bayer HealthCare LLC
511 Benedict Avenue
Tarrytown, NY 10591-5097 USA
914-631-8000